

Biodent EC40

tooth care varnish

Supports a healthy oral flora

When to use?

Biodent oral care products are used where there is frequent use of sugar and suboptimal brushing.

How often to use?

Use Biodent chlorhexidine varnish every 4 months.

Composition

Chlorhexidine, sandarac and ethanol.

Do not use

In case of hypersensitivity to chlorhexidine or any of the other ingredients.

www.biodent.nl • info@biodent.nl

BIODENT BV • Mollevite 19 • 6931 KE Westervoort
Nederland • T 0031 6 20143120





Instructions for use


With a blunt needle, apply a thin layer to the tooth surface that is difficult to clean with brushing. Avoid contact of chlorhexidine varnish with the tongue. Use Biodent varnish with chlorhexidine no more than $\frac{1}{4}$ applicator at a time.



Apply to the neck of the tooth



Apply to the chewing surface

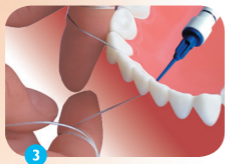

Apply between the teeth; floss



Do not eat or drink for 5 minutes after application; then, brush the varnish away with a toothbrush. Spit out small residual bits.

